

BLASTX[®]

Antimicrobial Biofilm Wound Gel


HOW TO APPLY

BLASTX[®] Antimicrobial Biofilm Wound Gel



Step 1 Cleanse

Genly cleanse the wound as directed by your healthcare professional.

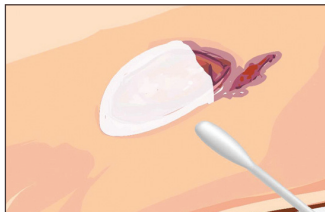

Step 2 Apply BlastX[®]

Apply BLASTX[®] directly to the wound, about 3 mm thick, covering to the wound edge.



Step 3 Cover

Cover with appropriate dressing. Reapply BLASTX[®] with each dressing change. Do not use alginates. Continue to use BLASTX[®] until healed.



ALWAYS READ THE LABEL AND FOLLOW THE DIRECTIONS FOR USE

Oraderm Pharmaceutical Pty Ltd (ABN 50 612 828 618),
15-17 Chapel St Cremorne 3121.
www.oraderm.com.au. Aug 23.



LEARN MORE